GEBRAUCHSINFORMATION

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USAGE INFORMATION

Please read this usage information carefully. It contains important information for you. In case you are unsure about proper product handling, please consult the manufacturer (see last page).

OVERVIEW

The LAMETEC IMPLAX® acupuncture implant is an approx. 2.5 mm long acupuncture needle made of titanium. It is designed to be implanted by any qualified doctor/therapist into the acupuncture reflex-points of the ear, to ensure a permanent effect.

To guarantee a safe application and for its protection during transport and handling, the implant is embedded in an implantation device (implantor).

PRODUCT SPECIFICATION

basic shape	cone, chamfered
length	2.5mm +/- 0.2 mm
diameter	0.8mm
material	medical grade II titanium
packaging	double sterile, for aseptic handling
applied by	accompanying Implantor
medical product	Yes, DIMDI Reg# DE/CA82/2004-002/01
classification	non-active implant IIb MDD
reusable	no
application range	ear implant acupuncture
package size	see batch marking
expires	see batch marking

APPLICATION RANGE

Implant Acupuncture provides a means to treat medical conditions and pathological influences which are in direct as well as indirect relation to the illness. Auricular acupuncture enhances the self-healing properties of the human body. The effects of Implant Acupuncture differ from those of conventional acupuncture mainly with the fact that the body's self-healing potential is stimulated permanently. The application of the implant triggers a continuous stimulus on the acupuncture points.

This process activates the body's information system – a cybernetic network of physical, mental and intellectual reactions – by means of electrical impulses. This, in return, stimulates the self-healing potential of the body and influences the immune defense and neural

transmitters. The permanent impulse provided by the implant will be controlled and regulated by the cybernetic network, intensified or deactivated if necessary – the result is harmonization.

SIDE EFFECTS

The permanent implantation of foreign substance into the bradytrophic tissue of the outer ear can cause inflammation which, in very rare cases, can lead to damage to the cartilage with severe cosmetic damage. The danger of inflammation increases considerably with repeated implantation.

INDICATIONS

It is recommended to apply the Implant for medical conditions where the therapy with conventional acupuncture has been approved.

CONTRAINDICATIONS

Caution! LAMETEC IMPLAX® should not be used in the following cases:

- Macumar intake
- Haemophilia
- Inflammation of the ear
- Short-term acute illnesses (may be treated with conventional acupuncture)

APPLICATION INSTRUCTIONS

The application is reserved for qualified therapists who have been trained to locate the relevant reflex points, about the effects of acupuncture and in the use of the LAMETEC IMPLAX® Implants.

The Implantation is carried out after professional disinfection of the ear to be treated. When it comes to determining the number of Implants to be applied, follow the principle "as few as possible, as many as necessary". This is to ensure that during a normal session between 4 and 12 Implants are being applied – depending on indication.

- Since the packaging is double sterilized, it is possible to pass the inner sterile packaging under aseptic conditions to the person applying it.

- The inner sterile packaging is taken out from the outer sterile packaging by ripping the packaging open.

- In order to take out the applicators, the thick end shall preferably be pressed through the paper of the inner sterile packaging.

PRECAUTIONS

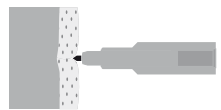
- The process of Implant ear acupuncture may only be carried out by qualified doctors/therapists.
- Do not use implants after their expiration date.
- Do not use implants whose inner sterile packing is damaged.
- The product must not be re-sterilized. »»

EN
USAGE (SCHEMATICALLY)


1 Hold the Implantor against the relevant acupuncture point of the ear. On-the-spot acupuncture is guaranteed.

2 Light pressure on the Implantor pushes the titanium Implant into the skin.

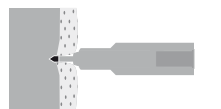3 Due to the small size of the Implant, the skin absorbs the needle immediately.

4 After approx. 2 days, the Implant is fully grown in at its destination point.

MANUFACTURER INFORMATION

LAMETEC Medizintechnik
Lutherplatz 10
08258 Markneukirchen
Germany
Hotline 0700 11 8 33 333 12ct/min
Phone +49 (0) 374 22 40 19 97
Fax +49 (0) 374 22 46 44 2
Email info@lametec.de
Internet www.lametec.de

FURTHER INFORMATION

If you need further information on the use of LAMETEC IMPLAX®, please visit our website www.lametec.de or contact Lametec by phone or email. LAMETEC always tries to ensure a high quality standard of its developments. We look forward to your suggestions, ideas and feedback anytime!

LAMETEC and IMPLAX are registered trademarks of LAMETEC Medizintechnik, Germany.

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